10S17

| | | |
|----|----------------------------------|----|
| DE | Gebrauchsanweisung | 2 |
| EN | Instructions for use | 6 |
| FR | Instructions d'utilisation | 10 |
| IT | Istruzioni per l'uso | 14 |
| ES | Instrucciones de uso | 18 |
| PT | Manual de utilização..... | 22 |
| NL | Gebruiksaanwijzing | 26 |
| SV | Bruksanvisning | 30 |
| PL | Instrukcja użytkowania | 34 |
| EL | Οδηγίες χρήσης | 38 |
| RU | Руководство по применению | 43 |

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Date of the last update: 2020-12-21

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- Please keep this document for your records.

1 Application and description

The Electric Wrist Rotator 10S17 is to be **exclusively** used for installation into arm prostheses. It controls the pronation and supination of the System Electric Hand or System Electric Greifer with quick-disconnect unit. Rotation of more than 360° is possible.

The 10S17 Electric Wrist Rotator is designed **exclusively** for use on one patient. Use of the product by other persons is not approved by the manufacturer.

The Electric Wrist Rotator can be operated in combination with the Myo Rotronic 13E205.

In case muscular tension is not sufficient, it may also be controlled by the Harness Pull Switch 9X14, the Traction Switch 9X18 or the Rocker Switch 9X25.

2 Safety instructions

Failure to follow the safety instructions given below can lead to damage to or malfunction of the product. Follow the safety instructions and the precautions specified in this accompanying document.

- Keep the product away from the immediate vicinity of active implantable systems (e.g. pacemakers, defibrillators etc.) since electromagnetic radiation may cause temporary interference.
- If the distance to HF communication devices (e.g. mobile phones, Bluetooth devices, WiFi devices) is too small, interference with internal data

communications can result in malfunctions of the electric wrist rotator. Maintaining the following minimum distances to these HF communication devices is recommended:

- Mobile phone GSM 850/GSM 900: 0.99m
- Mobile phone GSM 1800/GSM 1900/UMTS: 0.7m
- DECT cordless phones incl. base station: 0.35m
- WiFi (routers, access points,...): 0.22m
- Bluetooth devices (third-party products not approved by Ottobock): 0.22m
- Do not let foreign particles or liquids get into the device. The Electric Wrist Rotator should not be subjected to intense smoke, dust, mechanical vibrations, shocks or high temperatures.
- The Electric Wrist Rotator as well as any of its components may be opened or repaired only by Ottobock specialist staff.
- No liability will be assumed in case of possible damages if the device is operated incorrectly or used for purposes other than as originally intended.
- The Electric Wrist Rotator may only be operated with the EnergyPack Li-Ion battery 757B20 or 757B21, the 757B35=* MyoEnergy Integral or X-ChangePack 757B15.

3 Installation

3.1 Operation with MyoRotronic 13E205

Attach the MyoRotronic according to the Instructions for Use.

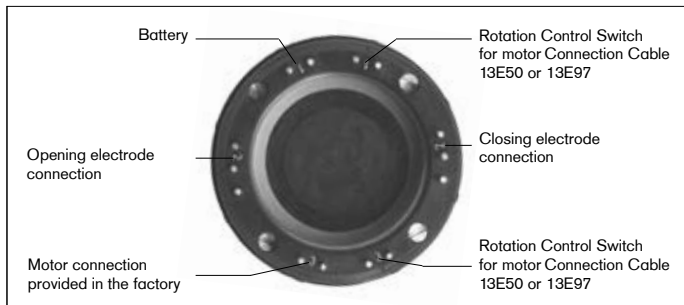
3.2 Operation by switch only

Connection pattern

Remove red protective plug and attach plugs numbered 1 and 3 of the Connection Cable 13E50 or the Connection Cable 13E97 with switch to the pins 1 and 3 of the Electric Wrist Rotator.

Attach Battery Connection Cable 13E51=2 or 13E51=4, or possibly, Battery Connection Cable 13E188=200 or 13E188=600 13E51=2 to the pins 1 of the Electric Wrist Rotator.

Attach the Electrode Cable 13E129 to the pins 2.

4 Installation into the prosthesis

Lubricate all cable plugs with Silicone Grease 633F11. After connecting all cables, use Assembly Tool 711M16 to engage the Electric Wrist Rotator into the lamination collar. Secure using Lock Ring 11S4.

5 Disposal

These products may not be disposed of with household waste in some jurisdictions. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

6 Technical data

| | | 757B15 | 757B20 | 757B21 | 757B35=* |
|------------------------------------|----------|--------|--------|--------|----------|
| Operating voltage | [V] | 6 | 7.2 | 7.2 | 7,2 |
| Rotation angle | [°] | >360 | >360 | >360 | >360 |
| No-load speed | r.p.m. | 13.5 | 17 | 17 | 17 |
| Corresponds to angular rotation of | [°/sec.] | 81 | 102 | 102 | 102 |
| Service life of the component | 5 years | | | | |

7 Symbols used

Medical device

8 Liability

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this device may only be used in combination with components that were authorised by the manufacturer (see instructions for use and catalogues). The manufacturer assumes no liability for damage caused by component combinations (use of third-party products) and applications which it did not authorise.

The device may only be opened and repaired by authorised Ottobock technicians.

9 Trademarks

All denotations within this accompanying document are subject to the provisions of the respective applicable trademark laws and the rights of the respective owners, with no restrictions.

All brands, trade names or company names may be registered trademarks and are subject to the rights of the respective owners.

Should trademarks in this accompanying document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

10 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices.

The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

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