453D2

DE Gebrauchsanweisung	2
EN Instructions for use	3
FR Instructions d'utilisation	5
IT Istruzioni per l'uso	6
ES Instrucciones de uso	8
PT Manual de utilização	9
SV Bruksanvisning	12
DA Brugsanvisning	14
FI Käyttöohje	16
PL Instrukcja użytkowania	18
HU Használati utasítás	20
CS Návod k použití	21
RO Instrucțiuni de utilizare	23
HR Upute za uporabu	24
SL Navodila za uporabo	25
SK Návod na používanie	27
BG Инструкция за употреба	28
TR Kullanma talimatı	30
EL Οδηγίες χρήσης	31
RU Руководство по применению	33
JA 取扱説明書	34
ZH 使用说明书	36
KO 사용 설명서	37

INFORMATION

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INFORMATION

Last update: 2020-07-22

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document for your records.

1 Indications for use

The Derma Seal Trans Ped 453D2 is to be **exclusively** used for the prosthetic fitting of amputations of the lower limb.

2 Indication

For partial foot amputee according to Lisfranc and Chopart.

3 Description

The Derma Seal Trans Ped is a stump sock of highly expandable Lycra®-spandex fabric. This seamlessly knitted stump sock offers fit without wrinkles to the foot stump. The distal area

inside the sock is provided with a polymer gel layer.

The polymer gel is extruded with a medical mineral oil, which has a beneficial effect on the skin. During wearing the oil will slowly diffuse from the gel and thus keep the skin smooth.

Due to the viscous nature of the gel, disturbing friction, pressure and weight bearing are reduced. The blended fabric of the sock keeps the skin dry and does not increase body temperature.

The Derma Seal Trans Ped offers high wearing comfort due to its fabric and the inner polymer gel layer.

4 Application and care

The Derma Seal Trans Ped stump sock is pulled over like a normal sock. The polymer gel should be placed directly on the skin of the residual limb. In this way its medical oil will ensure optimal care of the skin and keep it smooth.

After one day's use the stump sock should be washed by hand in warm water (approx. 30 °C) using a mild detergent. Thoroughly rinse with clear water. Do not wring.

The drying process in the area of the special polymer gel layer may last longer. To reduce the time of drying, the stump sock may be pre-dried in a towel immediately after washing. Then it should be laid flat on a drying frame or on a terry towel for complete drying.

The Derma Seal Trans Ped stump sock should not be worn in a wet condition.

Any tackiness of the polymer gel can be reduced by slightly dusting the gel layer with talcum powder.

For better hygiene of the residual limb skin it is recommended to remove the Derma Seal Trans Ped stump sock for a few hours each day, thus providing the skin with fresh air.

5 Size selection

Article number	Size	Length
453D2=N	Standard	22 cm
453D2=XL	extra long	50 cm

6 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

6.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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