8S511=* AxonSkin Silicone

DE Gebrauchsanweisung (Fachpersonal)	2
EN Instructions for use (qualified personnel)	9
FR Instructions d'utilisation (Personnel spécialisé)	16
IT Istruzioni per l'uso (Personale tecnico specializzato)	24
ES Instrucciones de uso (Personal técnico especializado)	32
PT Manual de utilização (Pessoal técnico)	39
NL Gebruiksaanwijzing (Vakmensen)	47
SV Bruksanvisning (Fackpersonal)	55
DA Brugsanvisning (Faguddannet personale)	62
NO Bruksanvisning (Fagpersonell)	69
FI Käyttöohje (Ammattihenkilöstö)	77
JA 取扱説明書 (有資格担当者)	84

INFORMATION

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INFORMATION

Last update: 2017-03-14

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The AxonSkin prosthetic glove is used as a protective glove for a prosthetic hand.

The multi-layered silicone material maintains its high elasticity even at low temperatures, thereby enabling constant functionality of the prosthetic hand.

8S511=* AxonSkin Silicone

The 8S511=* AxonSkin Silicone is designed for men.

Its appearance is based on a real hand in order to achieve a high degree of naturalness.

The prosthetic glove is coloured by hand, giving rise to possible variations in appearance, which are not a product defect.

The fingernails **cannot** be painted.

1.2 Combination possibilities

Designation	Reference Number
Prosthetic hand	8E500=* Michelangelo Hand

2 Intended use

2.1 Indications for use

The product may only be used as part of an upper limb prosthesis with Michelangelo Hand.

2.2 Area of application

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

2.3 Environmental conditions

Allowable environmental conditions
Allowable operating temperature: -10 °C to +60 °C (+14 °F to +140 °F)
Allowable storage temperature: -20 °C to +40 °C (-4 °F to +104 °F)
Allowable transport temperature: -20 °C to +60 °C (-4 °F to +140 °F)
Allowable relative humidity: max. 80%
Cleaning agents: warm water and soap

Unallowable environmental conditions
Mechanical vibrations or impacts, touching of sharp-edged or pointed objects
Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat (> +60 °C / > +140 °F)

2.4 Service Life

The product is a wear part, which means it is subject to natural wear and tear. The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal service life, in dependence of the level of use.

3 Safety

3.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General safety instructions

 CAUTION
Contact with cold, heat, embers or fire
Injuries (e.g. burns) and damage to the product due to unallowable environmental conditions
▶ Keep the product away from open flames, embers and other sources of heat.
▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").

 NOTICE
Unallowable combination of prosthetic components
Product damage due to malfunctions
▶ Only combine the product with prosthetic components that are approved according to the section "Combination Possibilities".

- ▶ Verify that the prosthetic components may be combined with each other by consulting their instructions for use.

NOTICE

Non-observance of allowable environmental conditions

Damage to product and soiling due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for soiling and damage.
- ▶ Clean the product immediately if it gets dirty, following the instructions in the section "Cleaning", in order to prevent permanent soiling.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ **Inform the patient.**

NOTICE

Contact with salves, lotions or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to salves, lotions or other products that contain oils or acids.

NOTICE


Careless use of the product

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the above mentioned objects.
- ▶ **Inform the patient.**

4 Scope of Delivery and Accessories


The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):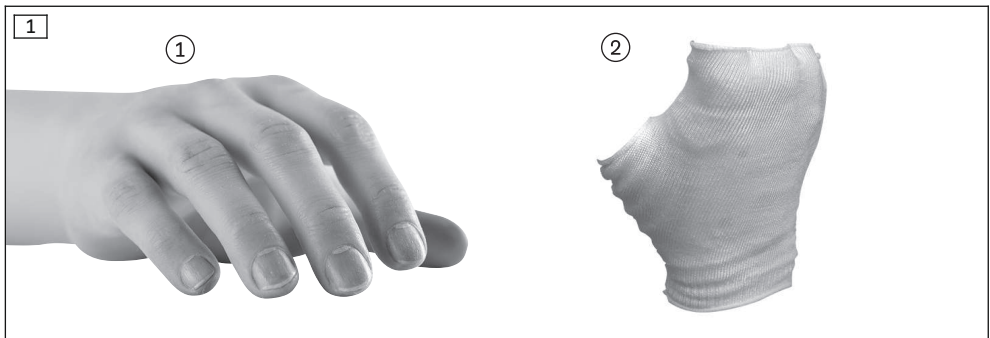

8S511=* AxonSkin Silicone scope of delivery					
Fig.	Item no.		Quantity	Designation	Article number
-	-	■	1	Instructions for use (qualified personnel)	647G861

8S511=* AxonSkin Silicone scope of delivery

Fig.	Item no.		Quantity	Designation	Article number
–	–	■	1	Instructions for use (patient)	646D678
1	①	–	1	Prosthetic glove	–
1	②	–	1	Textile glove	8Y29

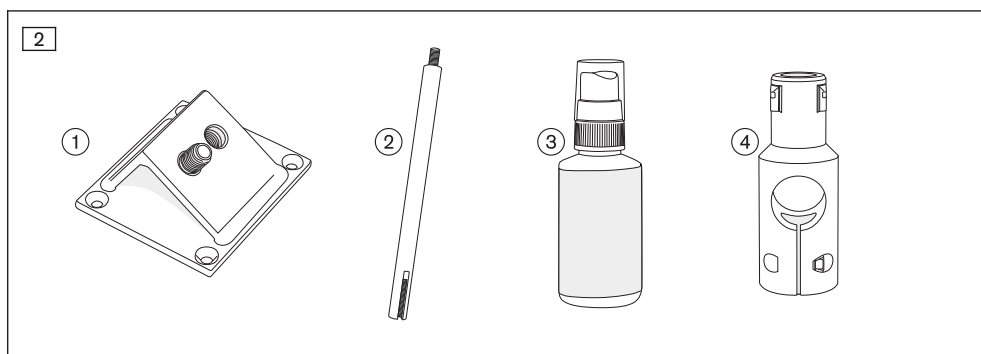Accessories

Fig.	Item no.		Quantity	Designation	Reference number
2	①	■	0	Mounting plate	711M2
2	②	■	0	Mounting tool (alignment rod)	711M1
2	③	■	0	Donning spray	640F18
2	④	■	0	Mounting tool	711M64

5 Preparation for use

NOTICE

Incorrect donning and removal of the prosthetic glove

Damage to the prosthetic hand and the prosthetic glove due to non-observance of instructions for use

- ▶ Observe all instructions for donning and removal. The prosthetic hand must always be mounted on the donning tool.
- ▶ Only use the components listed (equipment, tools, materials) and have them close to hand.
- ▶ Avoid damage that can be caused to the prosthetic hand and the prosthetic glove, e.g. from tools and excessive force (in particular on the thumb).

INFORMATION

It is not possible to colour the prosthetic glove with the 635P15=* Colour Pens.


5.1 Preparations for donning and removal

NOTICE


Non-observance of instructions for the donning tool

Damage to the product due to unsuitable donning tool


- ▶ Only use the donning tool specified and follow the donning instructions.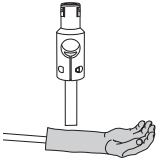

- > **Tools required:**
711M64 Mounting Tool, 711M1 Mounting Tool (Alignment Rod) and either 711M2 Mounting Plate or vice
- ▶ Install the mounting tool on the alignment rod.
INFORMATION: The alignment rod must be clamped in a vice, vertically for donning and horizontally for removal.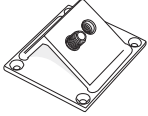

- ▶ **When using the mounting plate:**
INFORMATION: Install the mounting plate on a suitable surface and at optimal working height.
Attach the alignment rod to the mounting plate.

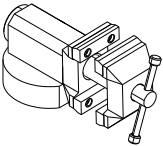- ▶ **When using a vice:**
Clamp the alignment rod into a vice at optimal working height.

- ▶ Set the prosthetic hand to hand position "Neutral position" (Neutral Mode) and lock the wrist.

- ▶ Install the prosthetic hand on the mounting tool.

5.2 Don the prosthetic glove on the prosthetic hand

INFORMATION

- ▶ Work quickly, adhering to the sequence of the instructions provided.


- ▶ **NOTICE! Never heat the prosthetic glove (e.g. with a blow drier or oven).**


- ▶ **NOTICE! Do not overstretch the textile glove during the donning process.**
Roll up the textile glove, slide it over the fingers of the prosthetic hand and unroll it over the prosthetic hand.

- ▶ Observe the correct positioning:
The cuff of the thumb is positioned in the last thumb groove.
The cuff of the wrist ends at the last groove in the hand shell.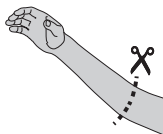


- ▶ Shorten the prosthetic glove to the desired length if necessary.

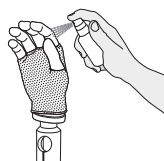- ▶ Only spray the fingers with donning spray.

- ▶ Spray the inside of the prosthetic glove with donning spray.

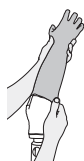- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the donning process.**
Position the prosthetic glove on the prosthetic hand for donning.

- ▶ Hold the prosthetic glove at the sleeve end and pull it over the prosthetic hand in one movement.


- ▶ **NOTICE! To prevent malfunctions, the prosthetic glove must not be twisted in the area of the fingers and especially the thumb.**
Adjust the prosthetic glove to fit the prosthetic hand properly.
- ▶ If required, adjust the length of the prosthetic glove to the prosthesis by shortening it.


5.3 Removal from the prosthetic hand

Removal with re-use of the prosthetic glove

- ▶ **NOTICE! Never heat the prosthetic glove (e.g. with a blow drier or oven).**

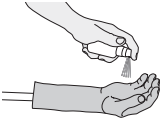
- ▶ Moisten the prosthetic glove with donning spray.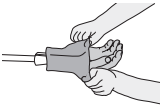


- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the removal process.**
Hold the prosthetic glove at the sleeve end and remove.

Removal without re-use of the prosthetic glove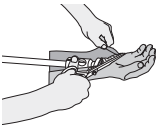

- ▶ **NOTICE! Avoid damaging the prosthetic hand during cutting.**
Cut open the prosthetic glove on the inside.

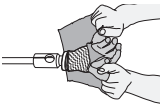- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the removal process.**
Pull the sleeve as far apart as possible and remove the prosthetic glove.

6 Use

6.1 Instructing the patient in the use of the product

INFORMATION

- ▶ Inform the patient about all the important points relevant to them that are contained in these instructions for use.

- ▶ Instruct the patient in the correct use of the product in accordance with the **Instructions for use (patient)** included in the scope of delivery.
- ▶ Hand over the Instructions for use (patient) to the patient when the briefing has been completed.

6.2 Cleaning

NOTICE

Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.
- ▶ **Inform the patient.**

> **Materials required:**

warm water, soap, cloth, lint-free cloth

- 1) Clean the product with warm water, soap and a cloth.
- 2) Rinse off soap residues with water.
- 3) Dry the product with a lint-free cloth and allow it to air dry fully.

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document (Instructions for Use). The manufacturer will not assume liability for damage caused by disregard of this document.

8.2 CE Conformity

This device meets the requirements of the 93 / 42 / EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to appendix VII of the guidelines.

9 Technical data

Article number Men's version	8S511=*
Sides	right (R), left (L)
Sizes	M
Colours	2, 4, 6, 8, 10, 12, 14

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Otto Bock HealthCare GmbH