Tube adapters

DE Gebrauchsanweisung	3	CS Návod k použití.....	33
EN Instructions for use	5	RO Instrucțiuni de utilizare.....	36
FR Instructions d'utilisation.....	7	HR Upute za uporabu	38
IT Istruzioni per l'uso	10	SL Navodila za uporabo	40
ES Instrucciones de uso	12	SK Návod na používanie	42
PT Manual de utilização.....	15	BG Инструкция за употреба	45
NL Gebruiksaanwijzing	17	TR Kullanma talimatı	47
SV Bruksanvisning	19	EL Οδηγίες χρήσης	50
DA Brugsanvisning	22	RU Руководство по применению	52
NO Bruksanvisning	24	JA 取扱説明書.....	55
FI Käyttöohje.....	26	ZH 使用说明书.....	57
PL Instrukcja użytkowania.....	28	KO 사용 설명서.....	59
HU Használati utasítás	31		

1

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-06-22

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

Tube adapters are used as components for modular lower limb prostheses. They connect the prosthetic foot to the proximal components. Adapter combinations allow for controlled angle and length adaptation in the sagittal and frontal plane as well as adjustment of inward and outward rotation. These instructions for use apply to the following tube adapters:

Diameter	Reference number
30 mm	2R2, 2R3, 2R30, 2R37, 2R38, 2R38=10, 2R49, 2R49=AL, 2R50, 2R50=AL
34 mm	2R36, 2R57, 2R58, 2R76, 2R77, 2WR95, 2WR95=1

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

Limited combination options for Ottobock components

- **2R30:** The tube can only be combined with 4R56* tube clamp adapters.
- **2R36:** The tube can only be combined with 4R156* tube clamp adapters.
- **2R30, 2R36:** Only use tubes proximal to the prosthetic knee joint.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

- The maximum approved body weight is specified in the technical data (see page 7).

2.3 Environmental conditions

Storage and transport

Temperature range -20 °C to +60 °C (-4 °F to +140 °F), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

2R2, 2R3, 2R38=10, 2R49, 2R49=AL, 2R50, 2R50=AL, 2R76, 2R77

Allowable environmental conditions

Temperature range: -10 °C to +45 °C (14 °F to 113 °F)

Moisture: relative humidity: 20% to 90%, non-condensing

Chemicals/liquids: fresh water as dripping water, occasional contact with salty air (e.g. near the ocean)

Solids: dust

Prohibited environmental conditions

Chemicals/moisture: salt water, perspiration, urine, acids, soapsuds, chlorine water

Solids: dust in high concentrations (e.g. construction site), sand, highly hygroscopic particles (e.g. talcum)

2R30, 2R36, 2R37, 2R38, 2R57, 2R58, 2WR95, 2WR95=1

Allowable environmental conditions

Temperature range: -10 °C to +45 °C (14 °F to 113 °F)

Chemicals/liquids: fresh water, salt water, perspiration, urine, soapsuds, chlorine water

Moisture: Submersion: max. 1 h in 3 m depth, relative humidity: no restrictions

Solids: Dust, occasional contact with sand

Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and damage (see page 6).

Unallowable environmental conditions

Solids: Highly hygroscopic particles (e.g. talcum), continuous contact with sand

Chemicals/liquids: Acids, continuous use in liquid media

2.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION!

Risk of injury and risk of product damage

- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 5).
- ▶ Note the combination possibilities/combinations exclusions in the instructions for use of the products.
- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ Observe the maximum lifetime of the product.
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Tube adapter	–
For 2R2, 2R3, 2R37, 2R38, 2R38=10		
4	Set screw	506G3=M8x12-V
For 2R49*, 2R50*, 2R57, 2R58, 2R76, 2R77, 2WR95*		
4	Set screw	506G3=M8x14

5 Preparing the product for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

5.1 Adjusting the adapter

CAUTION

Incorrect processing of tube

Fall due to damage to the tube

- ▶ Do not clamp the tube into a vice.
- ▶ To shorten the tube, use only a tube cutter or a cutting device.

- > **Required materials:** 719R3 tube cutter or 704Y14* cutting device, 718R1 tube deburrer
- 1) Shorten the tube to the required length.

- 2) Deburr the inside and outside of the cut edge with the tube deburrer.

5.2 Installation in the modular prosthesis

CAUTION

Incorrect mounting of the tube

Risk of injury due to breakage of load-bearing components

- ▶ Slide the tube all the way to the stop in the intended prosthetic component when mounting.

- > **Required materials:** See instructions for use of the connection components
- 1) Position the pyramid receiver of the tube adapter distally in the prosthesis.
- 2) Align the clamping slot of one tube clamp adapter:
Tube clamp adapter: **anterior**
Tube clamp adapter, sliding: **anterior** or **medial**
- 3) Connect the connection components to the tube adapter as described in the respective instructions for use.

Connecting the pyramid adapter and pyramid receiver

The pyramid adapter is fixed with the set screws of the pyramid receiver.

- > **Required materials:** torque wrench (e.g. 710D20), 636K13 Loc-tite 241
- 1) **Fitting:**
Screw in the set screws.
Use the torque wrench to tighten the set screws (**10 Nm**).
- 2) **Definitive assembly:**
Secure the set screws with Loctite.
Screw in the set screws.
Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten them (**15 Nm**).
- 3) Replace any set screws that are protruding or are recessed too much with appropriate set screws (see selection chart).

Selection table for set screws

Reference number	Length (mm)
506G3=M8X12-V	12
506G3=M8X14	14
506G3=M8X16	16

Alignment

The set screws in the pyramid receiver can be used to make static adjustments during alignment, trial fittings and after the prosthesis is finished.

Replacement and disassembly

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest and located next to each other.

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

2R2, 2R3, 2R37, 2R38, 2R38=10, 2R49, 2R49=AL, 2R50, 2R50=AL, 2R57, 2R58, 2R76, 2R77, 2WR95, 2WR95=1

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

10 Technical data

Tube diameter: 30 mm

Stainless steel		
Reference number	2R2	2R3
Weight [g]	195	315
Min. system height [mm]	97	
Max. system height [mm]	232	472
Min. build height [mm]	53	
Overall length [mm]	214	454
Max. body weight [kg]	100	136

Titanium			
Reference number	2R37	2R38	2R38=10
Weight [g]	160	275	
Min. system height [mm]	97		98
Max. system height [mm]	232	472	474
Min. build height [mm]	53		
Overall length [mm]	214	454	459
Max. body weight [kg]	100	136	100
Angling [°]	-		10

Aluminium			
Reference number	2R30	2R49	2R50
Weight [g]	200	255	155
Min. system height [mm]	69	97	
Max. system height [mm]	400	432	232
Min. build height [mm]	10	53	
Overall length [mm]	400	414	214
Max. body weight [kg]	100	125	

Aluminium		
Reference number	2R49=AL	2R50=AL
Weight [g]	255	155
Min. system height [mm]	97	
Max. system height [mm]	432	232
Min. build height [mm]	53	
Overall length [mm]	414	214
Max. body weight [kg]	136	

Tube diameter: 34 mm

Aluminium	
Reference number	2R36
Weight [g]	215
Min. system height [mm]	73
Max. system height [mm]	380
Min. build height [mm]	10
Overall length [mm]	380
Max. body weight [kg]	150

Titanium		
Reference number	2R57	2R58
Weight [g]	220	330
Min. system height [mm]	77	
Max. system height [mm]	282	472
Min. build height [mm]	27	
Overall length [mm]	264	454
Max. body weight [kg]	150	

Titanium		
Reference number	2WR95	2WR95=1
Weight [g]	330	
Min. system height [mm]	77	78
Max. system height [mm]	472	473
Min. build height [mm]	27	
Overall length [mm]	454	455
Max. body weight [kg]	150	
Angling [°]	-	6

Stainless steel		
Reference number	2R76	2R77
Weight [g]	260	370
Min. system height [mm]	77	
Max. system height [mm]	282	472
Min. build height [mm]	27	
Overall length [mm]	264	454
Max. body weight [kg]	150	

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