Otto Bock®

QUALITY FOR LIFE

CE

Otto Bock® Silicon ArmLiner



| 14Y1 |

Ⓓ	Patienteninformation	4
ⒼⒷ	Patient Information	6
Ⓕ	Information aux patients	8
Ⓘ	Informazione per l'amputato	10
Ⓔ	Información para el paciente	12
⒫	Informação para o paciente	14
Ⓐ	Patientinformation	16
ⒹⓀ	Patientinformation	18
ⒶⓁ	Patiënten informatie	20
⒫Ⓛ	Informacja dla pacjenta	22
Ⓕ	Pácienstájékoztató	24
ⒹⓇ	Hasta Bilgilendirme Broşürü	26
ⒼⓇ	Πληροφορίες για τον ασθενή	28

Otto Bock® Silicon ArmLiner

14Y1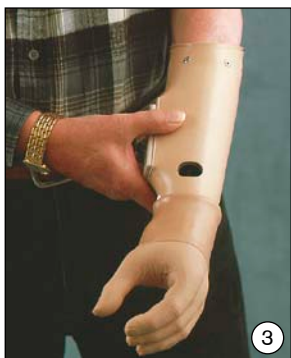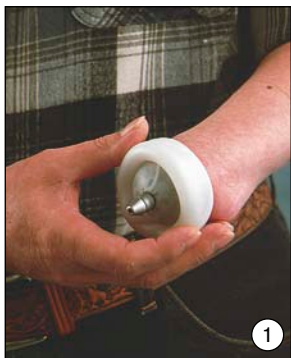

Patienteninformation

Otto Bock Silicon ArmLiner

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Patient Information

Otto Bock Silicone ArmLiner

The Otto Bock Silicone ArmLiner provides excellent suspension, increased comfort, and reduced shear forces. It is made of a bielastic material that offers good stability and durability.

When putting on your prosthesis while wearing the Otto Bock 14Y1 Silicone ArmLiner with pin, it will click audibly when the Otto Bock 14A110 Lock is engaged.

Putting on the Silicone ArmLiner

Attention

Do not apply the Silicone ArmLiner directly on injured skin!

Roll the Silicone ArmLiner all the way down and place your residual limb in the very end of the sleeve (fig. 1). Roll it up evenly and gently, being careful not to push too much fleshy tissue out of the top of the sleeve (fig. 2).

Attention

Make sure no wrinkles or air pockets are present!

Note:

It is important to roll the Silicone ArmLiner up gently; never pull it onto the residual limb!

Caution

When putting on the 14Y1 Silicone ArmLiner, be sure the pin is correctly positioned to lock onto the Otto Bock 14A110 Lock. You must hear it click into place (fig. 3).

With the arm prosthesis on (fig. 4), use the release button to disengage the lock between the 14Y1 Silicone ArmLiner and the prosthesis (fig. 5).

To take the Silicone ArmLiner off, roll the liner down (fig. 6).

Care Instructions

Attention

Careful daily care of both the liner and your skin are necessary to prevent skin irritation while wearing the Silicone ArmLiner.

To clean the Silicone ArmLiner, turn it inside out and use warm, running water and a neutral/pH-balanced, perfume-free soap. Dry the Silicone ArmLiner with a hand towel.

Note

Do not use any solvents such as thinner, acetone, benzene, etc. as these will damage the liner.

If there is significant perspiration, we recommend cleaning the Silicone ArmLiner and skin several times a day and applying a mild spray disinfectant.

Attention

Wait for the spray to dry before rolling the Silicone ArmLiner over the residual limb.

If the Silicone ArmLiner is still irritating your skin after approximately two weeks of wear, please contact your prosthetist or physician.

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- (S) **Otto Bock Scandinavia AB**
- (T) **Otto Bock South East Asia Co. Ltd.**
- (TR) **Otto Bock Ortopedi ve Rehabilitasyon Tekniği Ltd. Şti.**
Otto Bock HealthCare GmbH