How to use your new Rehband Support - 405436, 405336

SE

Funktion

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UK

Function

Gives support and warmth. Great flexibility to the knee joint. Protects the knee from blows.

Indications

Soft tissue overload, general soreness, general impact prevention.

Application

- 1) Place the support on the knee.
- 2) The curve of the lateral seam must be at the same level with the knee joint.

Important Information

- 1) If you have any questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/248°F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber* or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) SBR/neoprene products, time of use should not exceed 3-4 hours. Products made from other materials, time of use should not exceed 8 hours, unless it is recommended by the prescriber.
- 6) If the product contains natural rubber latex and you are allergic to natural rubber latex, we advise that you consult a specialist before using the support.
- 7) If you are allergic to heat, we advise you to use a product that does not contain SBR/neoprene.

* Medical- products, the prescriber must be a qualified doctor, physiotherapist, occupational therapist or orthopaedic engineer.

Washing Instructions

- 1) Mesh laundry bag is recommended.
- 2) Wash at 40 °C with similar colours.
- 3) Stretch product while wet and let dry.

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