
How to use your new Rehband Calf/Shin Sleeve - 106*, 7760*, 7960*

*different colors, sizes, thicknesses, sides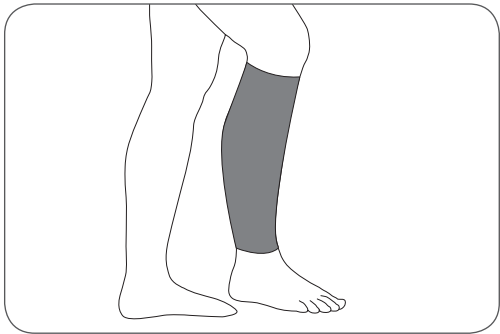

EN

Function

Gives support, compression and warmth to the shin. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides pain relief.

Indications

Periostitis, muscle rupture, over-exertion and soft tissue overload in the shin area. Pain, inflammation, wear and tear injuries.

Application

- 1) Pull the support up to the calf.
- 2) Tighten the Velcro fasteners at the front as required (7760*).

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer or the prescriber.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures (7760*).

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Conformity declaration

This product meets the requirements of European Directive 93/42/EEC for medical devices.

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene (see textile label)
Cover: polyamide and/or polyester (see textile label)

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