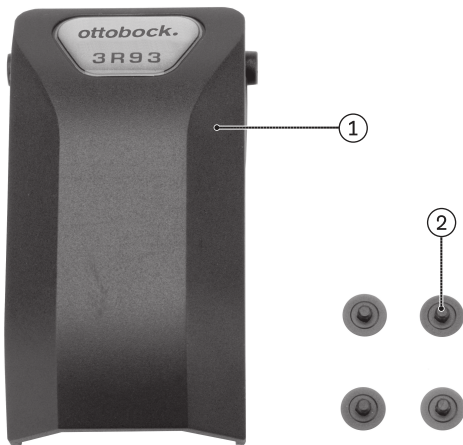
## 4D29

<b>DE</b> Kurzanleitung .....	6
<b>EN</b> Quick reference guide .....	7
<b>FR</b> Notice abrégée .....	8
<b>IT</b> Guida rapida .....	9
<b>ES</b> Instrucciones breves .....	10
<b>PT</b> Manual resumido .....	11
<b>NL</b> Beknopte handleiding .....	12
<b>RU</b> Краткое руководство .....	13

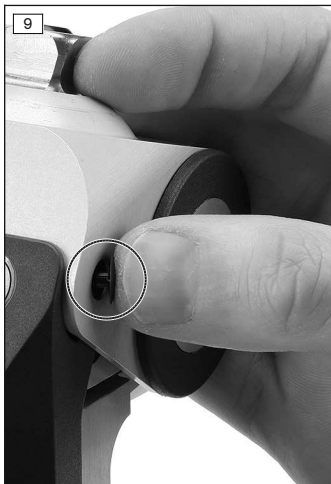
1

2

## INFORMATION

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## INFORMATION

Date of the last update: 2020-05-14

► Please read this document attentively.

The 4D29 Single Component Pack is used to replace the 4G550 sealing sleeve on the 3R93 Modular Friction Brake Knee Joint (see Fig. 1, Item 1). The installation of the sealing sleeve in order to prevent unintended separation of the same is described below. In particular, the proper installation of the 4G684 Safety Caps (see Fig. 1, Item 2) must be observed.

### Scope of Delivery

The scope of delivery for the product is shown on page 2 (Figure 1).

### Removing the Sealing Sleeve

- > You require the prosthetic knee joint and the 4D29 Single Component Pack.
  - 1) Fully flex the prosthetic knee joint.
  - 2) Entirely remove the safety caps still present in the prosthetic knee joint (see Fig. 2, Item 1).
  - 3) To remove the sealing sleeve, compress the tongues at the sides (see Fig. 3).
  - 4) Pull out the sealing sleeve (see Fig. 4).

### Installing the Sealing Sleeve

- 1) Push the sealing sleeve against the upper section, counteracting the spring force of the leg springs (see Fig. 5 and 6).
  - 2) The guide pins on the sealing sleeve have to engage in the bore holes in the upper section (see Fig. 7).
  - 3) The leg springs must engage in the grooves of the upper section (see Fig. 8).
  - 4) Push the safety cap into the bore hole in the sealing sleeve (see Fig. 9).
  - 5) Counter-support the inner pin of the sealing sleeve with a pair of pliers and carefully push the safety cap into the sealing sleeve to the stop (see Fig. 10).
  - 6) Check the sealing sleeve to ensure it moves freely. In order to do so, push the sealing sleeve away from the upper section (see Fig. 11) and verify whether it returns to the initial position independently due to the spring force (see Fig. 12).
- The installation of the sealing sleeve is now complete.

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