4023 Manu Rheuma 4024 Manu Rheuma Flex 4025 Manu Rheuma Pollex

DE Gebrauchsanweisung (Fachpersonal)	4
EN Instructions for use (qualified personnel)	8
FR Instructions d'utilisation (Personnel spécialisé)	12
IT Istruzioni per l'uso (Personale tecnico specializzato)	17
ES Instrucciones de uso (Personal técnico especializado)	21
PT Manual de utilização (Pessoal técnico)	25
NL Gebruiksaanwijzing (Vakmensen)	29
SV Bruksanvisning (Fackpersonal)	34
DA Brugsanvisning (Faguddannet personale)	38
NO Bruksanvisning (Fagpersonell)	42
FI Käyttöohje (Ammattihenkilöstö)	46
PL Instrukcja użytkowania (Personel fachowy)	50
HU Használati utasítás (szakszemélyzet)	55
HR Upute za uporabu (Stručno osoblje)	59
EL Οδηγίες χρήσης (Τεχνικό προσωπικό)	63
JA 取扱説明書 (有資格担当者)	67
ZH 使用说明书 (专业人员)	71


4023


4024

4025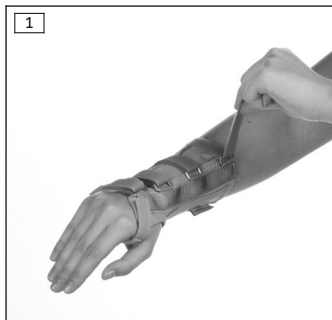

		Seite / Side	Größe / Size	Handgelenkumfang / Wrist circumference	
				cm	inch
				4023	L/R
L/R	XS	14,5	5.7		
L/R	S	15,5	6.1		
L/R	M	17,0	6.7		
L/R	L	18,5	7.3		
L/R	XL	20,0	7.9		
L/R	XXL	21,5	8.5		
4024 - 4025	L/R	XS	14,5	5.7	
	L/R	S	15,5	6.1	
	L/R	M	17,0	6.7	
	L/R	L	18,5	7.3	
	L/R	XL	20,0	7.9	
	L/R	XXL	21,5	8.5	

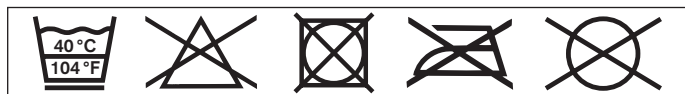Material	Darlexx®, PA, PU foam
-----------------	-----------------------

INFORMATION

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INFORMATION

Last update: 2015-06-24

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide important information on the fitting and application of the 4023 Manu Rheuma, 4024 Manu Rheuma Flex and 4025 Manu Rheuma Pollex wrist supports.

2 Application

2.1 Indications for use

The support is intended **exclusively** for treating the upper limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

2.2 Indications

4023 Manu Rheuma

- Rheumatoid irritation
- Osteoarthritis

4024 Manu Rheuma Flex and 4025 Manu Rheuma Pollex

- Rheumatoid irritation
- Tendovaginitis in the hand
- Osteoarthritis
- Sprains

Indications must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

Not known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent, swollen scars, reddening and hyperthermia in the relevant area; pronounced varicose veins, especially with impaired blood return, lymph drainage disorders and unclear soft tissue swelling distal to the support; sensory and circulatory disorders in the area of the forearm and hand, e.g. associated with diabetic neuropathy.

2.4 Effects

The support stabilises the wrist.

3 Safety

3.1 Explanation of Warning Symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General Safety Instructions

 **CAUTION**

Contact with heat, embers or fire

Injuries (e.g. burns) caused by melting of the material

- ▶ Keep the product away from open flame, embers and other sources of heat.

⚠ CAUTION**Reuse on other patients and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Only use the product for a single patient.
- ▶ Clean the product regularly.

⚠ CAUTION**Use of the product on skin sensitive to heat**

Skin irritation due to over-sensitivity to heat

- ▶ Do not use the product for more than 4 hours without a break.

NOTICE**Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

NOTICE**Contact with salves, lotions or other products that contain oils or acids**

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to salves, lotions or other products that contain oils or acids.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Size Selection

- 1) Determine the wrist circumference.

- 2) Determine the size of the support (see size chart).

4.2 Adaptation

CAUTION

Incorrect or excessively tight application

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the support is applied properly and fits correctly.
- ▶ Do not use the product in case of a known heat allergy.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

- > Open all hook-and-loop closures.
- 1) Remove plastic splints from the support.
- 2) Heat and adjust the plastic splints.
- 3) After they have cooled off slightly, insert the plastic splints in the support and close the hook-and-loop closures.

4.3 Application

- > Open all hook-and-loop closures.
- 1) Pull the support over the wrist, placing the loop over the thumb.
- 2) Fasten all hook-and-loop closures (see fig. 1 and see fig. 2).
- The support fits snugly without pinching.

4.4 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.
- 1) Take out all removable splints.
- 2) Fasten all hook-and-loop closures.
- 3) Recommendation: Use a laundry bag or net.
- 4) Wash the support in warm water at **40 °C** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 5) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).
- 6) Reinsert all splints.

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal Information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

6.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH