4X156-1/4X157-1/4X158-1

DE	Gebrauchsanweisung (Fachpersonal)	3
EN	Instructions for use (qualified personnel)	9
FR	Instructions d'utilisation (Personnel spécialisé)	14
IT	Istruzioni per l'uso (Personale tecnico specializzato)	21
ES	Instrucciones de uso (Personal técnico especializado)	27
PT	Manual de utilização (Pessoal técnico)	33
NL	Gebruiksaanwijzing (Vakmensen)	39
SV	Bruksanvisning (Fackpersonal)	45
DA	Brugsanvisning (Faguddannet personale)	51
NO	Bruksanvisning (Fagpersonell)	56
FI	Käyttöohje (Ammattihenkilöstö)	62
PL	Instrukcja użytkowania (Personel fachowy)	68
HU	Használati utasítás (szakszemélyzet)	74
CS	Návod k použití (Odborný personál)	80
SK	Návod na používanie (Odborný personál)	86
TR	Kullanma talimatı (Uzman personel)	92

INFORMATION

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INFORMATION

Date of last update: 2021-10-05

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The product "4X156-1 charger extension cable – ankle" is referred to below as product/charger extension cable – ankle.

The product "4X157-1 charger extension cable – knee" is referred to below as the product/charger extension cable – knee.

The product "4X158-1 charger extension cable – ankle, long" is referred to below as product/charger extension cable – ankle, long.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product description

The product permits the charger cable to be extended.

2.1 Combination possibilities

The product can only be combined with the following knee joints:

- 3C98-3, 3C88-3 C-Leg

3 Intended use

3.1 Indications for use

The product is to be used **exclusively** for relocating the charging receptacle of the 3C88-3 and 3C98-3 C-Leg electronic knee joint to the ankle area or the knee area.

3.2 Conditions of use

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

The product is intended **exclusively** for use on **one** patient. Use of the product by another person is not approved by the manufacturer.

4 Safety

4.1 Explanation of warning symbols

NOTICE

Warning regarding possible technical damage.

4.2 Structure of the safety instructions

NOTICE

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

NOTICE

Non-observance of safety notices

Damage to the product due to usage of the product in specific situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

NOTICE

Use of the product with damaged charger extension cable

Lack of proper charging functionality due to malfunction.

- ▶ When installing the charger extension cable, make sure there are no tight angles or kinks.
- ▶ Properly glue the charging receptacle to the joint.
- ▶ Properly glue the cable of the products 4X156-1/4X157-1/4X158-1 to the tube.

- ▶ Replace damaged charger extension cables immediately.

4.4 Patient information

NOTICE

Penetration of dirt and humidity into the product

Lack of proper charging functionality due to malfunction.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.

NOTICE

Mechanical stress on the product

Lack of proper charging functionality due to malfunction.

- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

5 Scope of delivery

- 1 pc. 4X157-1 charger extension cable – knee
 - 2 pc. 65X3 small O-ring
 - 1 pc. Instructions for use (qualified personnel)
- or
- 1 pc. 4X156-1 charger extension cable – ankle
 - 2 pc. 80X3 large O-ring
 - 2 pc. 65X3 small O-ring
 - 1 pc. 4X890 pad
 - 1 pc. Instructions for use (qualified personnel)
- or
- 1 pc. 4X158-1 charger extension cable – ankle, long
 - 2 pc. 80X3 large O-ring
 - 2 pc. 65X3 small O-ring
 - 1 pc. 4X890 pad
 - 1 pc. Instructions for use (qualified personnel)

6 Preparing the product for use

6.1 Alignment

6.1.1 Charger extension cable – knee

- 1) Open the charging receptacle cover on the knee joint.
- 2) Attach the charger extension cable to the knee joint.
- 3) Attach the charging receptacle of the charger extension cable to the frame of the knee joint.
- 4) Hook the O-ring into the bracket of the charging receptacle (see fig. 1).
- 5) Insert the counterpart piece of the charging receptacle into the O-ring.
- 6) Place the O-ring around the knee joint.
- 7) Hook the counterpart piece of the charging receptacle on the opposite frame side of the knee joint (see fig. 2).
- 8) Properly glue the cable to the joint (see fig. 3).
- 9) Properly glue the charging receptacle to the joint (see fig. 4).
- 10) Check the charging function of the knee joint.

6.1.2 Charger extension cable – ankle and long ankle

6.1.2.1 Preparation

- 1) Determine the position for the charging receptacle of the charger extension cable together with the patient.
- 2) Measure the distance between the flange of the charging receptacle and the outside of the cosmetic foam cover.
- 3) Clean the tube at the specified position with a damp cloth and mild soap (e.g. Ottobock 453H10=1 Derma Clean).
- 4) Prepare the pads if necessary.

6.1.2.2 Assembly

- 1) If necessary, fill the determined distance between the charging receptacle and the cosmetic foam cover with pieces of pad glued together one on top of the other (see fig. 5).
- 2) Open the charging receptacle cover on the knee joint.
- 3) Attach the charger extension cable to the knee joint.
- 4) If necessary, wrap the cable around the tube adapter.
- 5) Position the holder of the charging receptacle on the tube or, if necessary, on the pieces of pad that have been glued one on top of the other.
- 6) Secure the holder of the charging receptacle with the O-ring (see fig. 6).
INFORMATION: Use the small O-ring with up to 3 pad pieces glued on. Use the large O-ring with 3 or more pad pieces glued on.
- 7) Properly glue the cable to the tube.

- 8) Properly glue the charging receptacle to the joint (see fig. 4).
- 9) Check the charging function of the knee joint.

7 Disposal

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

8.3 CE conformity

The product meets the requirements of the RoHS Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic devices.

The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

9 Technical data

Environmental conditions	
Storage and transport in original packaging (≤3 months)	-20°C/-4°F to +40°C/+104°F

Environmental conditions	
Storage and transport without packaging (<48 hours)	-25°C/-13°F to +70°C/+158°F Max. 93% relative humidity, non-condensing
Long-term storage (>3 months)	-20°C/-4°F to +20°C/+68°F Max. 93% relative humidity, non-condensing
Operation	-10°C/+14°F to +60°C/+140°F Max. 93% relative humidity, non-condensing
Charging the battery	+10°C/+50°F to +45°C/+113°F
General	
Reference numbers	4X156-1, 4X157-1, 4X158-1
Product service life	Wear part subject to normal wear and tear

10 Appendices

10.1 Symbols Used

Declaration of conformity according to the applicable European directives

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

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Otto Bock Healthcare Products GmbH