
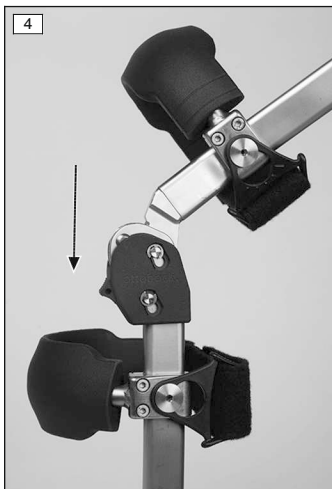
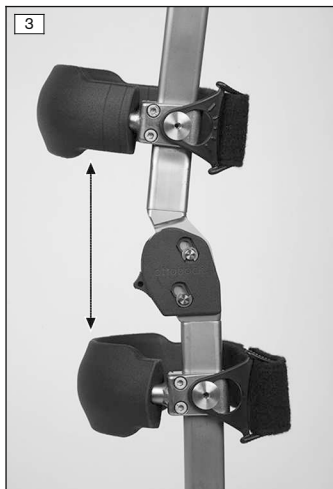
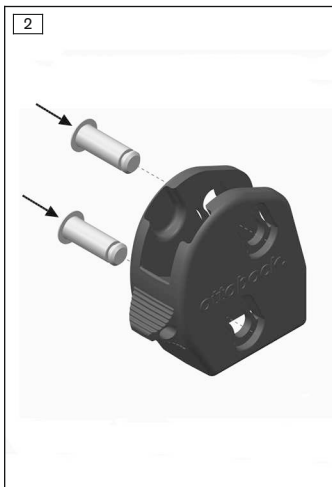
170D71

DE Montageanleitung	3
EN Assembly instructions	6
FR Instructions de montage	9
IT Istruzioni per il montaggio	12
ES Instrucciones de montaje	15
PT Instruções de montagem	19
NL Montagehandleiding	22
SV Monteringsanvisning	25
DA Monteringsvejledning	28
NO Monteringsanvisning	31
FI Rakennusohjeet	34
TR Montaj talimatları	37

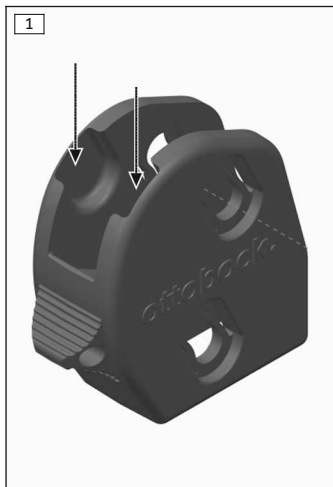INFORMATION

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INFORMATION

Date of last update: 2020-04-14

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the processing of the 170D71 3-phase switch.

2 Product overview

Component	Article number
3-phase switch	170D71
Lock axes	170D5=120-4
Lock rings	170Z7=4.0

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

3.2 General safety instructions

CAUTION

Incorrect alignment, assembly or adjustment

Risk of injury due to change in or loss of functionality

- ▶ Assembly, adjustment and maintenance operations may only be completed by qualified personnel.
- ▶ Observe the alignment, assembly and adjustment instructions.

CAUTION

Improper handling

Risk of falling due to accidentally opening or engaging the lock

- ▶ Avoid accidentally opening or engaging the lock.
- ▶ Instruct the patient in the correct handling of the orthosis joints.

4 Preparation for use

4.1 Installation of the 170D71 3-phase switch in the 170K1 Free Walk orthosis

- 1) Remove the joint protector.
- 2) Remove the lock rings from the two lower lock axes.
- 3) Push out the two lower lock axes.
- 4) Insert the 3-phase switch. In doing so, guide the groove in the 3-phase switch (see fig. 1) over the head of the bolt and push the 3-phase switch in the direction of the orthosis.
- 5) Insert the new lock axes into the orthosis knee joint from the medial side (see fig. 2).
- 6) Insert the lock rings from the lateral side.

4.2 Functional test

To verify technically correct functionality, take the foot section in one hand and the femoral section in the other:

Free Walk mode

- > The 3-phase switch is in the upper position.
- ▶ Push the orthosis ankle joint in the dorsiflexion direction and extend the orthosis knee joint.
 - The lock disengages automatically and the orthosis knee joint can be flexed.

Permanent locking

- > The knee joint is set to Free Walk mode.
- 1) Slide the 3-phase switch to the lower position with the knee extended.
- 2) Push the orthosis ankle joint in the dorsiflexion direction and extend the orthosis knee joint.
 - The knee joint is permanently locked.

Free motion mode

- > The knee joint is set to permanently locked mode.
 - 1) Slide the 3-phase switch to the upper position.
 - 2) Push the orthosis ankle joint in the dorsiflexion direction and extend the orthosis knee joint.
 - 3) Flex the orthosis knee joint.
 - 4) Slide the 3-phase switch to the lower position with the knee flexed.
 - The orthosis knee joint can be flexed freely at all times.
- In case of problems, please contact your Ottobock Free Walk specialist.

5 Cleaning

Promptly clean the product after contact with water containing salt, chlorine or soap, or if it gets dirty.

- 1) Rinse the product with clear fresh water.
- 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e. g. from an oven or radiator).

6 Maintenance

INFORMATION

The product may be exposed to increased loads by the patient.

- ▶ Shorten the maintenance intervals according to the expected loads.

7 Disposal

Dispose of the product in accordance with national regulations.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the

information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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