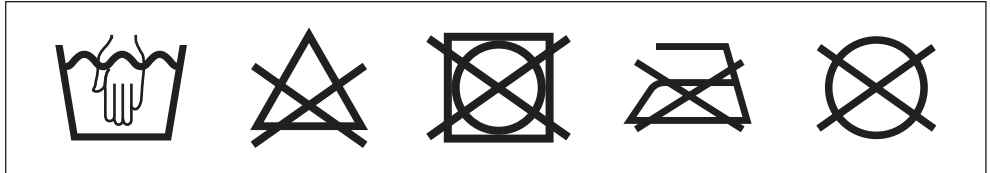
50S1 Dyna Ankle

DE Gebrauchsanweisung	4
EN Instructions for use	7
FR Instructions d'utilisation	11
IT Istruzioni per l'uso	15
ES Instrucciones de uso	18
PT Manual de utilização	22
NL Gebruiksaanwijzing	26
SV Bruksanvisning	29
FI Käyttöohje	33
PL Instrukcja użytkowania	36
HU Használati utasítás	40
CS Návod k použití	44
EL Οδηγίες χρήσης	47
RU Руководство по применению	51
ZH 使用说明书	55

Schuhgröße/Shoe size	
	35 - 37
	37 - 39
	39 - 41
	41 - 44

Material	PA, PE, PES, PP, PU, TPU, Stahl/steel
-----------------	---------------------------------------

INFORMATION

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INFORMATION

Date of last update: 2021-10-19

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 50S1 Dyna Ankle orthosis.

2 Intended use

2.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

2.2 Indications

- Conservative treatment of lateral ankle ligament ruptures
- Post-operative protection after suturing or reconstruction of the lateral ligaments of the upper ankle joint

2.3 Contraindications

2.3.1 Absolute Contraindications

Not known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb/body area; lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the medical device will be applied; sensory and circulatory disorders in the leg area.

- Condition following deep vein thrombosis of the leg and/or postthrombotic syndrome of the affected leg

2.4 Lifetime

The product is designed for a maximum lifetime of 6 months.

2.5 Effects

The orthosis sets the foot to slight pronation and dorsal extension. The dynamic restraint limits plantar flexion and supination while pronation and dorsiflexion movements remain unrestricted. The orthosis relaxes the anterior talofibular ligament.

3 Safety

3.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General safety instructions

 WARNING
Operation of motor vehicles Risk of accidents due to restricted body function <ul style="list-style-type: none">▶ Observe the applicable legal and insurance regulations for the operation of motor vehicles and have your driving ability examined and certified by an authorised agency.
 CAUTION
Reuse on other persons and improper cleaning Skin irritation, formation of eczema or infections due to contamination with germs <ul style="list-style-type: none">▶ The product may be used by one person only.▶ Clean the product regularly.

⚠ CAUTION**Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE**Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

NOTICE**Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

NOTICE**Use without suitable footwear**

Restriction of functionality due to insufficient stabilisation

- ▶ Only use the orthosis with closed footwear, taking the allowable effective heel height into account.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Size Selection

- ▶ Select the orthosis size based on the shoe size (see sizing table).

4.2 Fitting and Application

⚠ CAUTION**Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

⚠ CAUTION

Improper moulding or application

Injuries or damage to the orthosis due to overloading of the material and improper fit of the orthosis due to breakage of load-bearing components

- ▶ Do not make any improper changes to the orthosis.
- ▶ Always apply the orthosis according to the information in the instructions.

⚠ CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

- 1) **Optional: thermoforming the brace:** Thermoform the brace to adjust the fit. In order to do so, remove all hook and loop straps, heat the brace to **90 °C** max. and reshape it.
- 2) **Optional: attach additional pads with hook and loop:** In case of pressure sensitivity in the area of the ankle or shin, attach the additional pads with hook and loop straps (see fig. 1 and see fig. 2).
- 3) **Optional: apply anti-slip protection:** Clean the bonding surface in the area of the heel, roughen with fine sandpaper and attach the pimple rubber in the heel area using the supplied double-sided adhesive tape.
- 4) Open all hook and loop straps.
- 5) Set the foot into the brace. In doing so, ensure that the heel and forefoot are positioned comfortably.
- 6) Pull the middle hook and loop strap from the inside of the foot across the top of the foot and through the outer loop, and tighten it until the heel fits in the shell. Fasten the hook and loop strap in doing so.
- 7) Put the upper hook and loop strap through the loop and fasten it (see fig. 3).
- 8) Put the lower hook and loop strap through the loop and fasten it.
- 9) Stretch the functional strap and fasten it (see fig. 4).
- 10) Open the shoe wide, pull it over the applied brace and lace it.

5 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Remove the straps from the orthosis.
- 2) Fasten all hook-and-loop closures.
- 3) Hand-wash the straps and pads in warm water at **30 °C (86 °F)** using neutral detergent. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

6 Disposal

Dispose of the product in accordance with national regulations.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

7.2 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

7.3 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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