99B116 SoftTouch

DE Gebrauchsanweisung	2	CS Návod k použití.....	26
EN Instructions for use	3	RO Instrucțiuni de utilizare.....	27
FR Instructions d'utilisation.....	5	HR Upute za uporabu	29
IT Istruzioni per l'uso	7	SL Navodila za uporabo	31
ES Instrucciones de uso	9	SK Návod na používanie	33
PT Manual de utilização.....	11	BG Инструкция за употреба	35
NL Gebruiksaanwijzing.....	13	TR Kullanma talimatı	37
SV Bruksanvisning	15	EL Οδηγίες χρήσης	38
DA Brugsanvisning	16	RU Руководство по применению	40
NO Bruksanvisning	18	JA 取扱説明書.....	42
FI Käyttöohje.....	20	ZH 使用说明书.....	44
PL Instrukcja użytkowania.....	22	KO 사용 설명서.....	45
HU Használati utasítás	24		

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-03-31

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.

- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Function

The 99B116 SoftTouch cosmetic stocking is used as a cosmetic covering for a transtibial prosthesis. The cosmetic stocking is pulled over the foam cover. It improves the look of the prosthesis and provides added protection against splashed water.

1.2 Combination possibilities

The product must be used in TT prostheses only.

- The product is intended for use only with the 6R8 and 6R18 PE foam covers.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Lifetime

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General safety instructions

WARNING!

Risk of severe injuries

- ▶ Store the product out of reach of children. There is a risk of suffocation if the product is pulled over the mouth and nose.

CAUTION!

Risk of injury and risk of product damage

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to temperatures above +80 °C or below -20 °C.

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may only be used by one person; reuse on another person is prohibited.
- ▶ Observe the cleaning instructions.

NOTICE!

Risk of product damage and limited functionality

- ▶ Do not allow the product to come into contact with oils, ointments or lotions containing oils or acids.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	-
2	Cosmetic stocking	-

5 Preparing the product for use

NOTICE

Careless use of the product

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.

5.1 Assembling

- > **Prerequisite:** Remove all sharp objects from the hands and wrists (such as rings, bracelets or watches). Do not allow sharp fingernails to come into contact with the product.
- > **NOTICE! Proceed carefully and avoid pulling too hard on the stocking so it is not damaged.**
 - 1) Turn the stocking inside-out as far as the foot.
 - 2) Pull the stocking over the prosthetic foot so it fits correctly on the heel.
 - 3) Gradually pull the stocking over the prosthesis in the proximal direction, avoiding wrinkles.

6 Cleaning and Care

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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