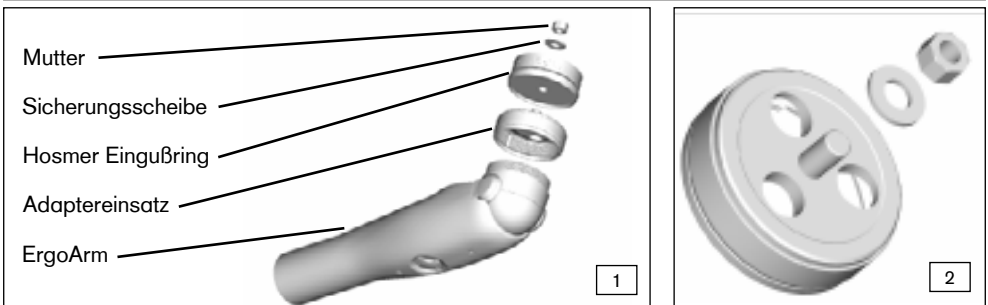
13Z68

DE Gebrauchsanweisung (Fachpersonal)	2
EN Instructions for Use (Qualified Personnel)	3

INFORMATION

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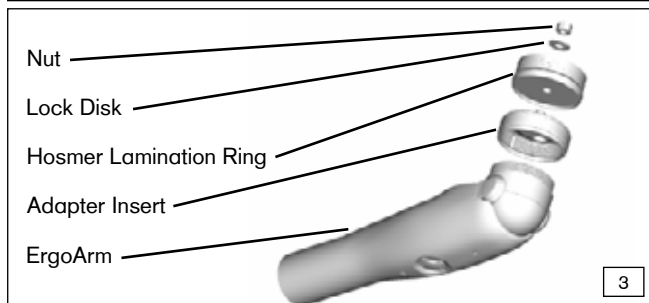
INFORMATION

Date of last update: 2013-01-10

> Please read this document carefully.

**1 Application**

The 13Z68 Adapter Insert is intended exclusively as a connection piece between an Above-Elbow Socket with a Hosmer Lamination Ring and an ErgoArm Elbow.

2 Description and Function

For initial fittings of an Above-Elbow Socket with a Hosmer Lamination Ring to an Elbow Component, use the 13Z68 Adapter Insert as a coupling. The Adapter Insert provides an easy and stable connection between the two prosthetic systems.

3 Assembly

1. Take off the prosthesis.
2. Remove the Hosmer Elbow and Forearm.
3. To correctly align the 13Z68 Adapter Insert, make sure the halfmoon shaped channel points in the direction of the hand (See Fig. 3)
4. Secure the assembly with the lock washer and nut.
5. Attach the ErgoArm Elbow as described in the component's Operating Manual.

4 Otto Bock Warranty

The OttoBock Warranty applies only if the product is used under specified conditions and intended purposes. The OttoBock warranty applies only to tested component combinations!

Attention:

The 13Z68 Adapter Insert is intended for a temporary fitting only. When creating a new socket, OttoBock recommends using the 13Z47 Lamination Ring

Otto Bock Healthcare Products GmbH
Kaiserstraße 39 · 1070 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com

Ottobock has a certified Quality Management Systems in accordance with ISO 13485.