
ottobock.

4R136=EL

DE	Gebrauchsanweisung	3
EN	Instructions for use	7
FR	Instructions d'utilisation	10
IT	Istruzioni per l'uso	14
ES	Instrucciones de uso	18
PT	Manual de utilização.....	21
NL	Gebruiksaanwijzing	25
SV	Bruksanvisning	28
DA	Brugsanvisning.....	32
NO	Bruksanvisning	36
FI	Käyttöohje	39
PL	Instrukcja użytkowania	43
HU	Használati utasítás	46
CS	Návod k použití	50
RO	Instrucțiuni de utilizare.....	53
HR	Upute za uporabu	57
TR	Kullanma talimatı.....	60
EL	Οδηγίες χρήσης	63
RU	Руководство по применению	67
JA	取扱説明書.....	71

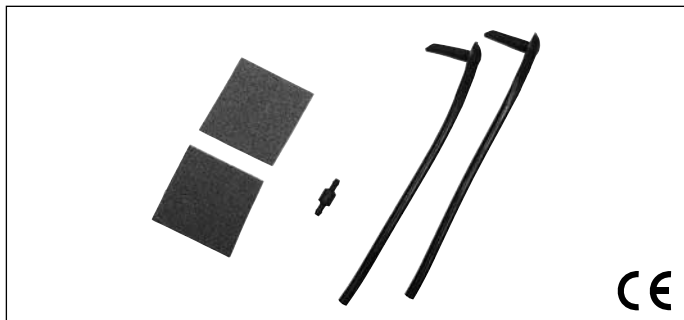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CAUTION Warnings regarding possible risks of accident or injury.

NOTICE Warnings regarding possible technical damage.

INFORMATION Additional information on the fitting / use.

INFORMATION

Last update: 2020-04-22

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

1 Description

1.1 Indications for use

The 4R136=EL Ottobock One-Way Valve is intended **exclusively** for use in prosthetic fittings of the lower limbs.

1.2 Field of application

In combination with a PUR- or TPE-coated knee sleeve, the 4R136=EL One-Way Valves serves to create a vacuum in a prosthetic socket.

1.3 Safety instructions

Please make your patients aware of the following safety instructions:

NOTICE

Risk of corrosion. Prosthetic components must not be exposed to corrosive elements such as fresh water, salt water and acids.

Using this medical device in such environmental conditions will render all claims against Otto Bock HealthCare null and void.

CAUTION

Risk of falling. Failure to observe the installation instructions can result in complete loss of vacuum and therefore significantly compromise the adhesion on the skin.

1.4 Function

The 4R136=EL One-Way Valve ejects excessive air out of the socket during the stance phase and closes during the swing phase to prevent further inflow of air. Thanks to this system, shearing and friction forces are effectively minimised.

2 Components

2.1 V4 EasyLine Valve Set (4R136=EL)

- V4 valve, 1 pc.
- Socket connector, 2 pcs
- Sandpaper, 2 pcs

3 Handling

3.1 Installation preparation

The following tools are needed in addition for installing the socket connector:

- Long nose pliers
- 6 mm drill bit
- Isopropyl alcohol
- 2-component epoxy adhesive (5 min. curing time)
- Knife with a round blade
- Insulating or polyethylene tape

3.2 Completion

Laminate the socket in the usual way. Drill a 6 mm bore hole into the posterior side of the socket, as close to the distal end as possible (Fig. 1). Remove any burrs caused by drilling.

Slightly roughen the inner and outer area of the bore hole (bonding area) with the enclosed sandpaper. Clean the bonding area with isopropyl alcohol. Do the same with the socket connector.

Mix the 2-component epoxy adhesive. Apply a thin layer of adhesive to the flat section of the angle piece as well as to the neck of the tapered part of the connector (Fig. 2).

When applying the adhesive, be careful not to let any adhesive get into the open end of the connector.

INFORMATION

In case of very long or narrow sockets, insert the pointed connector through the bore hole, make a mark on the inner side with a pencil, take out the connector and cut it off according to the mark. This version may require that the bore hole is a bit larger (7 mm). Then apply the adhesive as described above.

Insert the connector through the hole so that the flat angle piece is pressed against the outer side of the socket. Secure this connection with adhesive tape and let cure (Fig. 3).

Using a knife with a round blade, cut off the part of the connector that is protruding inside the socket (Fig. 4) so that it is flush with the socket wall (Fig. 5).

Then mount the 4R136=EL One-Way Valve to the end of the hose.

4R136=EL: The arrow on the valve shows the direction of air flow (Fig. 6).

4 Care instruction

The valve can be cleaned with the aid of compressed air.

5 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

5.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard

of this document, particularly due to improper use or unauthorised modification of the product.

5.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

5.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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