4R170=1, 4R170=2

DE Gebrauchsanweisung	5	CS Návod k použití.....	56
EN Instructions for use	9	RO Instrucțiuni de utilizare.....	60
FR Instructions d'utilisation.....	12	HR Upute za uporabu	64
IT Istruzioni per l'uso	17	SL Navodila za uporabo	68
ES Instrucciones de uso	21	SK Návod na používanie	72
PT Manual de utilização.....	25	BG Инструкция за употреба	76
NL Gebruiksaanwijzing.....	29	TR Kullanma talimatı	80
SV Bruksanvisning	33	EL Οδηγίες χρήσης	84
DA Brugsanvisning	37	RU Руководство по применению	88
NO Bruksanvisning	41	JA 取扱説明書.....	92
FI Käyttöohje.....	44	ZH 使用说明书.....	96
PL Instrukcja użytkowania.....	48	KO 사용 설명서.....	99
HU Használati utasítás	52		

1

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-03-15

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

With the 4R170=1 and 4R170=2 sliding adapters, the flexion position of a prosthetic socket can be adjusted by up to 4° on an arc. The adapters consist of a concave base plate and a sliding attachment with threaded connector. The attachment with threaded connector can be replaced by the 4R173 attachment with pyramid receiver.

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Approved for a body weight of **up to 150 kg**.

The product may only be used in **TF** prostheses **proximal** to the prosthetic knee joint.

2.3 Environmental conditions

Storage and transport

Temperature range -20 °C to +60 °C (-4 °F to +140 °F), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

Allowable environmental conditions

Temperature range: -10 °C to +45 °C

Moisture: relative humidity: 20% to 90%, non-condensing

Unacceptable environmental conditions

Chemicals/liquids: fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

Solids: dust, sand, highly hygroscopic particles (e. g. talcum)

2.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General safety instructions

 **CAUTION!**

Risk of injury and risk of product damage

- ▶ Observe the maximum lifetime of the product.

- ▶ To avoid the risk of injury, observe the instructions for assembling screw connections: Use correct screws, clean threads prior to assembly, observe tightening torques, observe instructions for thread lock
- ▶ Note the combination possibilities/combination exclusions in the instructions for use of the products.
- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 9).

NOTICE!

Risk of product damage and limited functionality

- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

4R170=1, 4R170=2 sliding adapters		
Quantity	Designation	Reference number
1	Instructions for use	–
1	Adapter	–
For the assembly of a socket adapter (4R22, 4R23, 4R37, 4R51, 4R54, 4R55, 4R74, 4R77, 4R95)		
4	Countersunk screw	501S41=M6x12-12.9

4R170=1, 4R170=2 sliding adapters		
Quantity	Designation	Reference number
For the 4R73=A or 4R73=D socket adapter, use the 501S41=M6x12x11.4 countersunk screws included with the socket adapter.		

Spare parts/accessories (not included in the scope of delivery)	
Designation	Reference number
Attachment with pyramid receiver	4R173

5 Preparing the product for use

CAUTION

Incorrect alignment or assembly

- Risk of injury due to damaged prosthetic components
- ▶ Observe the alignment and assembly instructions.

5.1 Sliding Adapter Preparation

The attachment with external thread can be replaced with the attachment with pyramid receiver. Connection with a shuttle lock system is possible through the pyramid receiver. Both attachments are equipped with a locking disk.

Optional: Locking disk alignment (4R170=2 only)

The locking disk for the sliding attachment must be aligned by turning it by 180°, so as to achieve the maximum displacement path.

> Required tools and materials:

710D21 Torque Wrench

- 1) Loosen the cap screw on the sliding attachment (see fig. 3).
- 2) Align the locking disk so that maximum displacement can be achieved (see fig. 4).
- 3) Tighten the cap screw (assembly torque: **25 Nm**).

Optional: Changing the sliding attachment

> Required tools and materials:

710D21 Torque Wrench

- 1) Unscrew the cap screw from the sliding attachment, then remove the attachment (see fig. 5).
- 2) Place the other attachment on the base plate while aligning the locking disk in the groove (see fig. 6).
- 3) Screw the cap screw into the attachment, then tighten it (assembly torque: **25 Nm**).

5.2 Mounting the Sliding Adapter

INFORMATION

When the sliding adapter is placed 300 mm under the socket reference point, the change in the length of the prosthesis as the socket flexion angle changes is negligible.

Attachment with thread

> Required tools and materials:

710D21 Torque Wrench, degreasing cleaner (e.g. 634A3 Acetone)

- 1) Use a degreasing cleaning agent to clean the thread of the attachment and that of the connection component before final assembly.
- 2) Screw the sliding adapter completely into the connection component.
- 3) **INFORMATION: The sliding adapter must be aligned in the sagittal plane.**
Unscrew the sliding adapter from the connection component to a maximum alignment of **360°**.
- 4) Follow the instructions for the connection component (especially with regard to assembly torque) to properly close its threaded clamping.

Attachment with pyramid receiver

> Required tools and materials:

710D21 Torque Wrench, 636K13 Loctite® 241

- ▶ Align the sliding adapter in the sagittal plane.

1) For final assembly:

Use Loctite® to secure the set screws.

- 2) Screw in the set screws.

3) For trial fitting:

Use the torque wrench to tighten the set screws (**10 Nm**).

For final assembly:

Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten (**15 Nm**).

- 4) Replace any set screws that are protruding or recessed too much with suitable ones (see selection table).

Selection Table for Set Screws

Reference number	Length (mm)
506G3=M8x12-V	12
506G3=M8x14	14
506G3=M8x16	16

Replacement and Disassembly

When replacing or disassembling the prosthesis component, the previously adjusted position can be maintained if the two deepest screwed-in set screws, which are NOT located opposite to each other, are removed.

5.3 Mounting a Socket Adapter to the Sliding Adapter

Mounting

> Recommended tools and materials:

710D21 Torque Wrench, 636K13 Loctite® 241

- ▶ Follow the instructions for your selected socket adapter to properly mount it to the four-hole connector of the sliding adapter (assembly torque: **12 Nm**) (see fig. 7).

5.4 Sliding Adapter Adjustment

> Tools required:

710D21 Torque Wrench

- 1) Loosen the cap screw on the sliding attachment of the sliding adapter by **1 turn**.
- 2) Set the desired angle (see fig. 8).
- 3) **CAUTION! Snap the locking disk of the sliding attachment into the groove to prevent the adapter from twisting.**
Tighten the cap screw (assembly torque: **25 Nm**).

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

10 Technical data

Reference number	4R170=1	4R170=2	4R173
Weight [g]	550	440	220
System height [mm]	15		42
Build height [mm]	15		17
Material	Stainless steel		
Adjustment range [°]	4		–
Max. body weight [kg]	150		

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