2Z500, 2Z501

DE Gebrauchsanweisung	5
EN Instructions for use	10
FR Instructions d'utilisation	16
IT Istruzioni per l'uso	22
ES Instrucciones de uso	28
PT Manual de utilização	34
NL Gebruiksaanwijzing	40
SV Bruksanvisning	45
DA Brugsanvisning	51
NO Bruksanvisning	57
FI Käyttöohje	62
HR Upute za uporabu	68
JA 取扱説明書	73

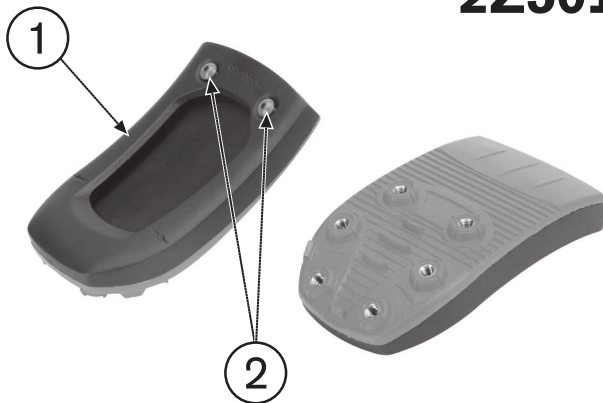
1

2Z500

2

2Z501

3

4

5

INFORMATION

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INFORMATION

Date of last update: 2020-07-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

1.1 Construction and Function

The 2Z500 or 2Z501 Outer Soles are mounted on the 1E90 Sprinter Prosthetic Foot. The 2Z501 Outer Sole with spikes is used for running on an all-weather running track while the 2Z500 Outer Sole without spikes is for all other surfaces.

1.2 Combination Possibilities

Designation	Reference Number
Prosthetic foot	1E90 Sprinter

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Approved for a body weight of up to **125 kg**.

2.3 Environmental conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing
Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

2.4 Service life

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General safety instructions

 **CAUTION**

Excessive strain on the product

Risk of injury due to breakage of load-bearing components

- ▶ Use the product according to the specified area of application (see Page 11).

 **CAUTION**

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

 **CAUTION**

Exceeding the service life and reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Ensure that the approved service life is not exceeded.
- ▶ Only use the product on a single patient.

 **NOTICE**

Mechanical damage to the product

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).

- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

Signs of changes in or loss of functionality during use

Movement and noises between the prosthetic foot and the outer sole, as well as a lack of surface grip are noticeable signs of a loss of functionality.

4 Scope of Delivery

2Z500 Outer Sole without spikes					
Fig.	Item No.		Quantity	Designation	Reference Number
–	–	■	1	Instructions for Use	647G848
1	1	–	1	Outer sole without spikes	–
1	2	▲	2	Oval head screw	501S72=M5x14-1

2Z501 Outer Sole with spikes					
Fig.	Item No.		Quantity	Designation	Reference Number
–	–	■	1	Instructions for Use	647G848
2	1	–	1	Outer sole with spikes	–
2	2	▲	2	Oval head screw	501S72=M5x14-1
–	–	■	1	Spike key	710Z3
–	–	▲	6	Spikes	–

5 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

5.1 Mounting the Outer Sole

> Required tools and materials:

710D4 Torque Wrench, degreasing cleaner (e.g. 634A58), 636K13 Loctite® 241

- 1) Clean the mounting surface on the prosthetic foot and, if necessary, on the outer sole too with a degreasing cleaner.
- 2) Mount the outer sole onto the prosthetic foot (see fig. 3).
- 3) Secure the screw connection with Loctite.
- 4) Screw the outer sole on to the prosthetic foot with two oval head screws (see fig. 4) (mounting torque: **4 Nm**).

5.2 Installing the Spikes (only for 2Z501)

> Required tools and materials:

710Z3 Spike Key

- Use the spike key to screw the spikes into the thread on the outer sole and tighten by hand (see fig. 5).

6 Use

CAUTION

Incorrect use of prosthetic foot and outer sole

Risk of injury due to lack of surface grip and damage to the product

- Only use the prosthetic foot with an outer sole.
- Only use outer soles with spikes on sports floors suitable for this purpose.

The 2Z500 Outer Sole is suitable for recreational running sports on various surfaces. The outer sole has a multilayered construction and adapts to bends and uneven surfaces. It features a low-wear, self-cleaning tread.

2Z500 Outer Sole without spikes

Suitable for:	Hard surfaces (e.g. asphalt, park paths, wood paths)
Not suitable for:	Smooth surfaces (e.g. ice, tiles), soft surfaces (e.g. sand, gravel, mud)

The 2Z501 Outer Sole with spikes is suitable for recreational and competitive sport on relevant surfaces. Worn spikes can be replaced individually.

2Z501 Outer Sole with spikes

Suitable for:	Sports surfaces fit for spikes (e.g. all-weather running track)
Not suitable for:	Smooth or hard surfaces (e.g. ice, tiles, concrete, asphalt), soft surfaces (e.g. sand, gravel, mud)

7 Cleaning

- 1) Clean the product with a damp cloth and soap.
- 2) Rinse the product with clean, fresh water.
- 3) Dry the product with a soft cloth.
- 4) **NOTICE! Do not store the product while moist.**
Allow to air dry in order to remove residual moisture.

8 Maintenance

CAUTION

Failure to follow the maintenance instructions

Risk of injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the following maintenance instructions.
- ▶ Arrange regular maintenance intervals with the patient depending on the level of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Direct the patient to observe the following maintenance instructions:
Check the entire prosthesis for damage after each use.
In case of changes in or loss of functionality and damage, do not continue using the prosthesis and have it checked by qualified personnel.
Comply with the maintenance intervals.

9 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

10 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

10.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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