3R65

Ⓓ	Otto Bock® Modular-Kniegelenk für Kinder, einachsig, mit hydraulischem Schwunghasenregler	3
ⒼⒷ	Otto Bock® Modular Single Axis Knee Joint for Children with Hydraulic Swing Phase Control	6
Ⓕ	Articulation modulaire de genou Otto Bock® pour enfants, monoaxiale, avec régulateur hydraulique de la phase oscillante	9
Ⓘ	Ginocchio modulare Otto Bock® per bambino, monoassiale, con regolatore idraulico	13
Ⓔ	Articulación de rodilla modular Otto Bock® para niños, monoaxial, con regulador hidráulico	16
ⒶⒻ	Enkelassig modulair Otto Bock® kniescharnier voor kinderen, met hydraulische zwaafaseregelaar	19
Ⓐ	Otto Bock® Modul-knäled för barn, enaxlig, med hydraulisk svingfaskontroll	23
ⒸⒶ	Μονοαξονική δομοστοιχειωτή άρθρωση γόνατος της Otto Bock® για παιδιά, με υδραυλικό έλεγχο στη φάση αιώρησης	26

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English

1.1 Field of Application and Purpose

The 3R65 Modular Knee Joint is **exclusively** suitable for the prosthetic fitting of children.

It is a monocentric (single axis) joint with integrated miniaturised hydraulic unit. **For its use the limits have been established as follows: 45 kg (99 lbs.) body weight, 145 cm body height and foot size 21 cm.**

Connection to proximal (to the socket) is provided by means of the pyramid adapter, to distal it takes place by means of the clamp attachment in the lower joint section where the modular tube is inserted.

1.2 Design

Upper and lower joint sections are constructed of light metal and are connected to one another by a specially designed axis. For swing phase control, a miniaturised hydraulic unit is integrated into the joint body. It serves to create dynamic resistances to movement that optimise the gait pattern by preventing excessive heel rise during flexion and terminal impact during extension.

The resistance to movement (damping behaviour) can be easily changed by adjusting the cross section of flow. The overall level of resistance to knee flexion and knee extension may be independently adjusted.

- The linkage geometry allows powerful resistance to knee flexion (heel rise) throughout the range of swing phase motion.
- Extension resistance may be set to a low value, since it automatically increases gradually as the knee approaches full extension.

Unlike mechanical gait controls, the hydraulic unit automatically compensates for changes in the amputee's walking speed, to ensure that the prosthesis moves smoothly and naturally at all times.

The flexion angle of approx. 135° is limited by the modular adapter hitting the lower joint section (Fig. 1).

2 Safety Instructions

- The 3R65 Modular Knee Joint may not be used for bathing prostheses.
- Please follow all indicated torque values!
- Use the Patient Information section of this pamphlet (Section 3) to inform your patient about the function and use of the Modular Knee Joint.

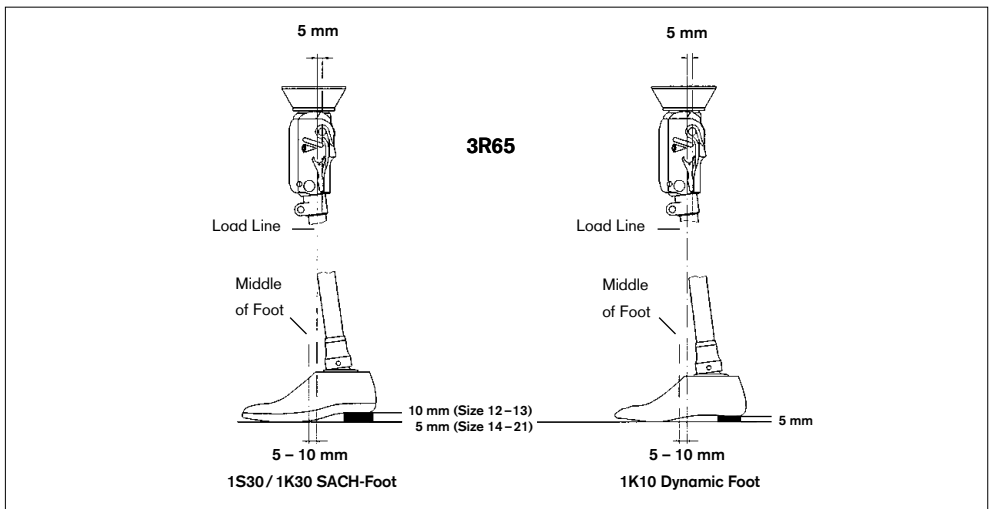
3 Patient Information

- The modular knee joint should not be subjected to intense smoke, dust, mechanical vibrations, shocks or high temperatures.
- Do not allow foreign particles or liquids to get into the modular knee joint. Avoid exposing prosthetic components to corrosive elements such as fresh water, salt water, acids and other liquids. Using this medical device in such environmental conditions will render all claims against Otto Bock HealthCare null and void.
- The modular knee joint as well as damaged components may be opened or repaired only by certified Otto Bock Service technicians.
- The knee joint function may be affected by both environmental and service conditions. To reduce risk for the patient, the knee joint should not be used when noticeable functional changes occur. Noticeable functional changes may include poor response, incomplete extension, diminished swing phase control or stance phase stability, abnormal noises, etc. If you notice any damage or functional changes, please have the prosthesis inspected by a specialist workshop.
- When using the knee joint do not reach into the joint mechanism since there is the risk of injury.

4 Alignment

The starting point for achieving correct alignment is to determine the individual load line and to transfer the measurement data to the prosthetic components. Knee stability during standing and during the stance phase of walking is obtained by posterior placement of the knee axis by approx. 5 mm (see illustration).

We recommend the combination with the 1K10 Dynamic Foot or the 1S30/ 1K30 SACH-Foot.

5 Adjustment of Swing Phase Control

To avoid damaging the knee, adjust swing phase control using the 4X50 Special Key that is provided attached to the knee! Once the knee has been adjusted, the prosthetist should keep this key for your future use. Resistance to flexion and extension are separately adjusted by moving pins on each side of the knee. The adjustment pins are only accessible in the slots **with the knee flexed**.

5.1 Adjustment of Swing Phase Flexion Resistance (fig. 2)

Flex the knee up to the stop. Insert the special key provided into the slot **on the right side** and adjust following the scale:

lowest flexion resistance = 0

highest flexion resistance = 8

Remove the key from the slot with the knee still flexed.

5.2 Adjustment of Swing Phase Extension Resistance (fig. 3)

Flex the knee up to the stop. Insert the special key provided into the slot **on the left side** and adjust following the scale:

lowest extension resistance = 0

highest extension resistance = 8

Remove the key from the slot with the knee still flexed.

6 Directions for Installation

Selection of the proper tube adapter depends on foot size:

Article no.	for foot sizes	tube length	body weight
2R41=2 (red)	up to 17 cm	260 mm	of max. 35 kg / 77 lbs.
2R41=1 (black)	18 to 21 cm	300 mm	of max. 45 kg / 99 lbs.

Insert the tube adapter until it reaches the bottom of the tube clamp.

Attention!

Never pull the 2R41 Tube Adapter out of the clamp attachment in an effort to adapt the length of the prosthesis to the growth of the child. Instead, always cut a new tube adapter to the appropriate length and insert it into the tube clamp until it stops.

Torque value of clamping screw: 3 Nm (2.2 ft. lbf. / 26.5 lbf.in.).

Torque value of adjustment screws of the adapters: 9 Nm (6.5 ft. lbf. / 80 lbf. in.)

Use 710D4 Torque Wrench.

Secure the screws by means of 636K13 Loctite.

7 Cosmetic Foam Cover

Use the 3S48 Foam Cover for Knee Joint 3R65.

Attention!

Do not use talcum powder to eliminate noise in the cosmetic foam cover. Talcum powder reduces the lubrication of the mechanical parts, which may lead to a malfunction and thus increase the risk of failure. Using this medical product after application of talcum powder will render all claims null and void.

Note:

To reduce friction and to eliminate noise, please apply 519L5 Silicone Spray directly on the contact surfaces of the cosmetic foam cover.

8 Maintenance Instructions

**Attention!**

Do not disassemble the joint! In case of any problems please return the entire joint to Otto Bock.

Please examine the modular knee joint after it has been in use for a few weeks as well as every three months and make any necessary adjustments. Whenever servicing the knee, replace the Rubber Stop.

8.1 4D17 Single-Component-Pack

Consisting of 1 oval head screw, 1 rubber stop.

9 Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the Instructions for Use. The device must only be operated with tested modular components in accordance with the Otto Bock Mobility System. The manufacturer is not liable for damage caused by component combinations that were not authorised by the manufacturer.

10 CE Conformity

This product meets the requirements of the 93/42/EEC guidelines for medical devices. This product has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock Healthcare Products GmbH