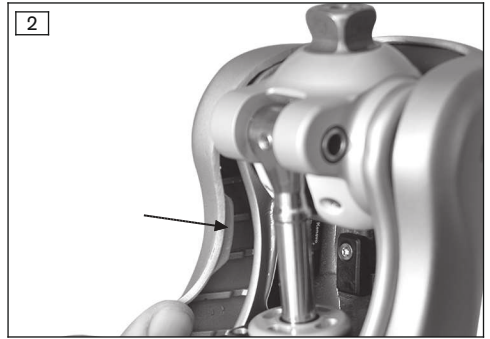
## Kenevo Protector 4X840

<b>DE</b> Gebrauchsanweisung (Benutzer) .....	3
<b>EN</b> Instructions for use (user) .....	7
<b>FR</b> Instructions d'utilisation (Utilisateur) .....	10
<b>IT</b> Istruzioni per l'uso (Utilizzatore) .....	14
<b>ES</b> Instrucciones de uso (Usuario) .....	19
<b>PT</b> Manual de utilização (Usuário) .....	23
<b>NL</b> Gebruiksaanwijzing (Gebruiker) .....	27
<b>SV</b> Bruksanvisning (Användare) .....	30
<b>DA</b> Brugsanvisning (Bruger) .....	34
<b>NO</b> Bruksanvisning (Bruker) .....	38
<b>FI</b> Käyttöohje (Käyttäjä) .....	42
<b>PL</b> Instrukcja użytkowania (Użytkownik) .....	46
<b>HU</b> Használati utasítás (felhasználó) .....	50
<b>CS</b> Návod k použití (Uživatel) .....	54
<b>SK</b> Návod na používanie (Užívateľ) .....	58
<b>TR</b> Kullanma talimatı (Kullanıcı) .....	62
<b>RU</b> Руководство по применению (Пользователь) .....	66

## INFORMATION

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## INFORMATION

Date of last update: 2021-03-01

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The product “4X840 Kenevo Protective Cover” is referred to simply as the Protective Cover/product below.

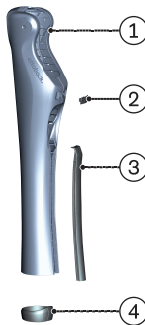
These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

## 2 Product description

### 2.1 Design

The product consists of the following components:

1. Protective cover main unit
2. Upper closure (one side always stays on the main unit)
3. Lengthwise closure
4. Flexible cover cap

### 2.2 Function

The protective cover provides a means of enclosing the Kenevo electronic knee joint, including the tube adapter, to enhance its appearance and function. The housing of the knee joint is protected against external influences by the protective cover, e.g. scratching and surface contamination. The protective cover is not required for the proper and safe functioning of the joint.

## 3 Intended use

### 3.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 3.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

### 3.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

## 4 Safety

### 4.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

### 4.2 Structure of the safety instructions

#### The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

### 4.3 General safety instructions

#### Non-observance of safety instructions

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety instructions and the stated precautions in this accompanying document.

#### Improper disassembly, assembly and handling of the product

- > Injury due to pinching of body parts.
- > Possible injury in case of parts with sharp edges.
- ▶ Use increased caution during disassembly/assembly and handling of the product.

#### Changes or modifications to the product made independently

Injury due to faulty operation or malfunction of the product.

- ▶ Have any changes or modifications to the product carried out only by authorised, qualified Ottobock personnel.

## 5 Scope of delivery

- 1 pc. Kenevo Protective Cover main unit
- 1 pc. 4P840=R lengthwise closure
- 1 pc. 4P860=U upper closure
- 1 pc. 4P8 Distal cap
- 1 pc. Instructions for use (user)

## 6 Handling

### INFORMATION

Removing the flexible cover cap from the prosthetic foot is not possible. The flexible cover cap remains loose on the prosthetic foot when the protective cover is not being used.

### 6.1 Protective Cover Installation

#### NOTICE

#### Loss of stability due to inadequately installed retainers

No hold of the Protective Cover on the knee joint.

► Ensure that all retainers and closures are properly engaged and/or fastened.

- 1) Open or remove the closures.
- 2) Spread open the protective cover.
- 3) Put the protective cover over the knee joint and apply it (see fig. 1).

**INFORMATION: The retainers of the protective cover (at the side in the upper closure area) have to engage on the frame element of the knee joint (see fig. 2).**

- 4) Squeeze the protective cover together with both hands.
- 5) Fasten the upper closure (see fig. 3).
- 6) Fasten the lengthwise closure by sliding it in.
- 7) Align the two lengthwise grooves on the flexible cover cap with the lengthwise closure (see fig. 4).

### 6.2 Removing the Protective Cover

- 1) Pull down the flexible cover cap until the protective cover is exposed.
- 2) Open or remove the closures.
- 3) Spread open the protective cover and remove it from the knee joint.
- 4) Fasten the two closures after removing the protective cover.

## 7 Cleaning and Care

#### NOTICE

#### Improper product care

Damage to the product due to the use of incorrect cleaning agents.

► Only clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## 8 Legal information

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices.

The full text of the regulations and requirements is available at the following Internet address:  
<http://www.ottobock.com/conformity>

### 8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

## 9 Technical data

Ambient conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F max. 93% relative humidity, non-condensing
Operation	-10 °C/+14 °F to +40 °C/+104 °F max. 93% relative humidity, non-condensing

General	
Reference number	4X840
Product service life	Wear part subject to normal wear and tear
Protective cover weight (with closures)	380 g
Flexible cover cap weight	11 g

## 10 Appendices

### 10.1 Symbols Used

Manufacturer

Declaration of conformity according to the applicable European directives

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