

Liability/Warranty

This device is warranted for a period of 90 days. The manufacturer’s warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use. Details of the warranty are also available at www.ottobockus.com.

Manufacturer’s Limited Warranty

Otto Bock HealthCare, LP (Ottobock) warrants all of its devices, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the devices are used as intended, without unapproved modifications, following all Ottobock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners. This Limited Warranty does not cover device damage caused by accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Ottobock, or any alteration or repair by others that, in Ottobock’s judgment, materially or adversely affect the device.

The duration of this Limited Warranty varies by product types and is effective from the date of delivery to the end-user. Please refer to www.ottobockus.com or call 800 328 4058 for questions. Ottobock’s sole obligation under this Limited Warranty shall be to repair, replace, refabricate the device at no charge, or refund the cost of the device to the original purchaser, at Ottobock’s sole discretion.

THE EXPRESS WARRANTIES SET FORTH ABOVE ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTOBOCK. IN NO EVENT SHALL OTTOBOCK’S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTOBOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion of incidental or consequential damages, thereby rendering the aforementioned limitation in applicable to certain original purchasers.

Ottobock Walker Boot
Instructions for Use



50S159=* Ottobock Walker Boot

Thank you for using this Ottobock device. The following information will help you fit and care for this device. Please review it before using this device and ask for clarification if needed.

Indications for Use:

Indicated for Achilles tendon ruptures, forefoot or midfoot fractures, ankle fractures, distal fibula fractures, soft tissue injuries, post-operative use, trauma and rehab.

Warning

This device does not prevent injury and is not intended to reduce or eliminate the risk of injury.

Warning

Always consult with your physician or healthcare provider before making changes to the brace.

Warning

Carefully read all instructions for use before using or fitting this product.

Warning

Proper rehabilitation and activity modification are also a part of a safe treatment program.

Warning

If you experience pain, swelling, or any unusual reaction, contact your physician or healthcare provider immediately.

CAUTION

This device is offered under order of a physician or other qualified healthcare provider.

CAUTION

This device is intended for single patient use and is not intended to be reused on a second patient.

Application Instructions

1. Unfasten straps by disengaging the hook and loop fasteners on the strap end and pulling the strap through the d-ring.
2. Remove the dorsal panel and set aside.
3. With the patient in a seated position, unfasten the softgood liner and place foot inside the boot. Secure the softgood liner using the hook and loop closure and ensuring the pneumatic pump is at the front of the tibia (shin bone).
4. To adjust the toe guard, push the toe guard up and remove. Reapply to desired length if needed.
5. Position the anterior shell on the front of the leg. Ensure the pneumatic bulb and release valve fit within the predesigned slots on the anterior shell.
6. Secure the straps. Starting at the ankle, pull the straps tight, loop through the d-ring, and secure using the hook and loop fastener. Repeat for the remaining calf strap. Then secure the forefoot strap.
7. Inflate the boot. Press the pneumatic bulb repeatedly on the front side of the shin to inflate the boot. Frequently check with the patient to ensure the bladder does not over inflate. Use the release valve to deflate the bladder if needed. **NOTE: Do not over inflate the Air Bladder. Overinflating the bladder may stop blood circulation to the patient's foot.**

Cleaning and Care

Remove softgood liner and wipe boot shell clean with a damp cloth. Softgood liner can be hand washed with mild detergent. Triple rinse, squeeze and air dry. Avoid twisting the material to remove water.

Clinical Fitting Instructions

Select size according to shoe size.

Adult Sizing Guide

Size	Men's	Women's	Euro
Small	4.5 - 7	6 - 8	36 - 38
Medium	7.5 - 10.5	8.5 - 11.5	39 - 42
Large	10.5-12.5	11.5-13.5	43 - 46
XLarge	12.5+	13.5+	47+